

## Lurocoat® Ophthalmic Viscoelastic Contraindications & Precautions

### **CONTRAINDICATIONS**

At present, there are no contraindications to the use of LUROCOAT® when used as recommended.

### **PRECAUTIONS**

**CAUTION: Injection of viscoelastics creates pressure in the syringe. To prevent expulsion of the cannula into the eye, ensure that the cannula is securely attached to the Luer-Lok™ fitting on the syringe.**

Observe the usual precautions taken during anterior segment surgery.

Pre-existing glaucoma or the surgery itself can cause increased intraocular pressure after the procedure(4). The following precautions should be carefully considered.

! Do not use excessive amounts of LUROCOAT®.

! Remove LUROCOAT® from the anterior chamber at the end of the procedure.

! Corrective therapy should be initiated if the postoperative intraocular pressure rises above safe levels.

## Ortholure™ Orthopedic Viscosupplement Contraindications & Precautions

### **CONTRAINDICATIONS**

Do not administer Ortholure to patients with known hypersensitivity (allergy) to sodiumhyaluronate, hyaluronan preparations or to one of the other constituents.  
Do not administer Ortholure to patients with cases of infections or skin diseases in the area of the injection site.  
Do not administer Ortholure to patients if the synovial joint is severely inflamed or infected.  
Do not administer Ortholure where joint replacement is not feasible or appropriate.

### **PRECAUTIONS**

Caution should be exercised in patients with known hypersensitivity to drugs.  
General precautions for intra-articular injections should be observed.  
As with any invasive joint procedure there is the rare possibility of infection.  
Do not inject Ortholure intravascularly as it may cause adverse systemic events.  
Do not inject Ortholure extra-articularly.  
Ortholure should be used with caution in patients with evidence of lymphatic or venous stasis in the leg.  
The effectiveness of a single treatment cycle of less than 3 injections of Ortholure has not been established.  
Do not use Ortholure if the prefilled syringe or sterile package is opened or damaged. Store in the original packaging at room temperature 2-25 °C.  
Do not freeze!  
Do not use after the expiry date indicated on the box and labels.  
Keep out of the reach of children!